
MRS Duo

molift® by Etac

EN - User manual

BM20001 Rev. C 2021-06-01

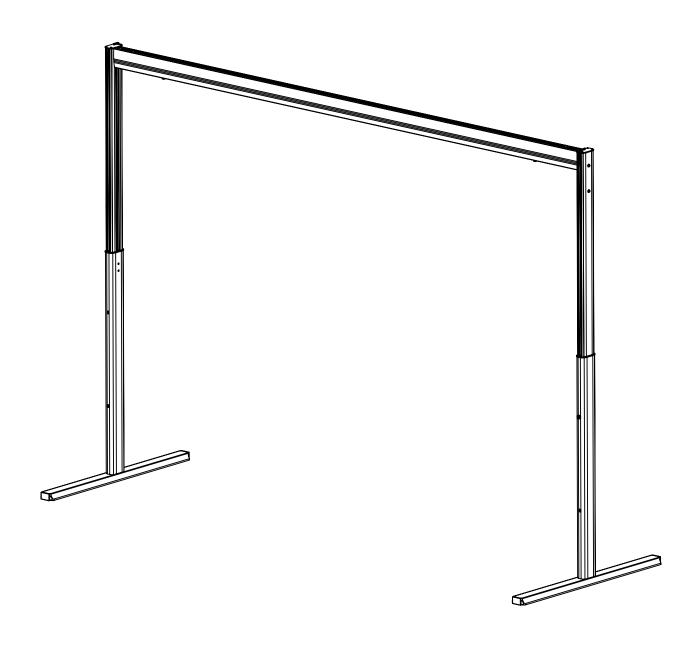



English Manual

Content

MRS Duo	2
MRS components	2
About MRS Duo	2
General	
Declaration of conformity	3
Conditions for Use	3
Warranty	
Product identification	3
Technical Data	
Assembly	
Supplied parts	
Mounting foot	
Mounting boom to post	
Adjusting post heights	
End stops	6
Mounting charger to post	
Check list before use	
How to use MRS Duo	
Lifting user	
Transport	
Transport and storage	7
Accessories	8
Molift AIR	
Molift Nomad	
Boom	8
Maintenance	9
Cleaning and disinfection	
Reconditioning	
Recycling	
Spare parts	
Periodic Inspection	
Pericodic Inspection Scope	
Troubleshooting	

Important

This User Manual contains important safety instructions and information regarding the use of the MRS Duo and accessories.

In this manual, the user is the person being lifted. The assistant is the person operating the lifter.



Warning!


This symbol indicates important information related to safety. Follow these instructions carefully.



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

MRS Duo

MRS Duo components:

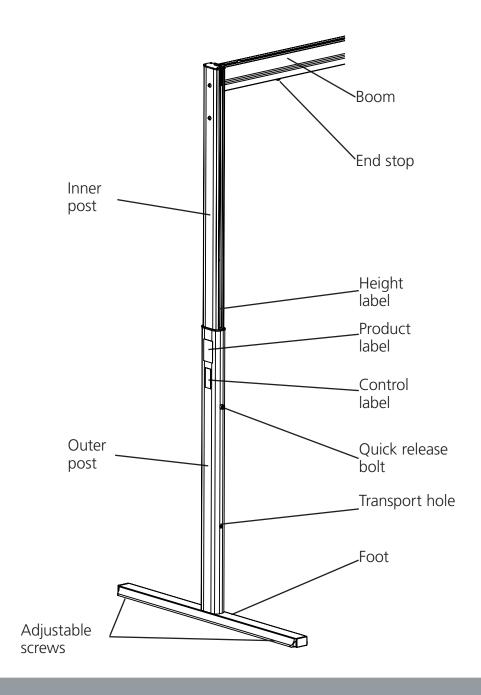

About MRS Duo

MRS Duo is a mobile 2 post gantry intended for Molift Air and Molift Nomad lifters. The purpose is for lifting and transferring a person from/to bed, floor, chair, wheelchair or toilet with a sling. The MRS Duo is made of light-weight materials, can easily be disassembled and mounted for new locations, and has four step adjustable posts from 2,25 to 2,55 meters. Boom ranges from 2 to 3,5

meters and the lifting capacity is 300 kg (661 lbs). It is ideal for hospitals and institutional, in environments where permanent installations are not possible or desirable, but can also be used in privates homes. The MRS Duo should only be used indoor on flat surfaces. Such as bedroom, bathroom, living room etc.

General



CE declaration

The product, and its related accessories, described in this instruction for use (IFU), is in compliance with the regulation (EU) 2017/745 of 5. April 2017 – as a medical device, risk class I.

Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Conditions for Use

Lift and transfer of a person will always pose a certain risk and only trained personnel are allowed to use the equipment and accessories covered by this user manual.

Modifications and use of components made by other manufacturers.

We recommend only using Molift components and spare parts. Declaration of conformity is not valid and Etac is not responsible for warranty if any modifications are made to the product. Etac shall not be liable for faults or accidents that can occur when using components made by other manufacturers.



Only certified personnel are allowed to perform repairs.

Warranty

2-year warranty against defects in workmanship and materials of our products.

Product identification

Product label

The Product label contain article number and production date.






Warning labels and Symbols

Symbols used on the product, explained in more detail:



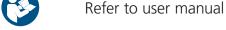



Date of manufacture

Manufacturer

REF Catalogue number

SN Serial number

MD Medical Device

Technical Data

Safe Working Load (SWL)

300 kg (661 lbs)

Weight of unit

Post (complete): 14,8 kg/30,9 lbs Boom Kg/m: 4,2 kg/m/8,8 lbs/m

Foot: 3,8 kg/6,6 lbs

Post (without foot): 11 kg/24,3 lbs Total weight (2 posts and boom): 38-44,3 kg/83,8-97 lbs

Available boom lengths (A):

2m - 2,5m - 3m - 3,5 m

Dimensions:

Width (D): 2,15 - 3,65 m

Height (B): (4 steps x 100mm) 2,25-2,55 m

Depth (E):1,2m

Max internal space (AxC): 3,5x2,4m


Transport height: 1,7 m


Material:

Anodized aluminum, Plastic

Expected Lifetime:


The MRS Duo has an expected lifetime of 10 years

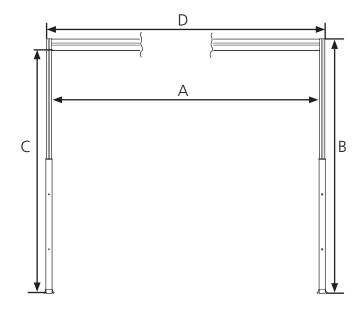

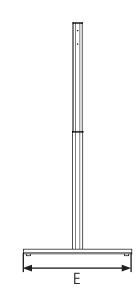

Assembly

Supplied parts

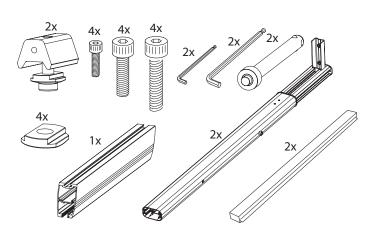


Make sure the following parts are included in the package.



Beware of sharp edges at the ends of the boom.

Mounting Foot

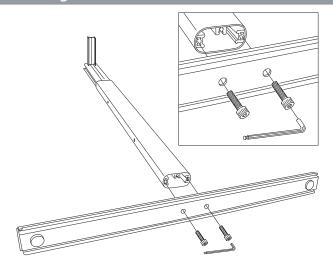


- 1. Lay post on the floor.
- 2. Use the longest M10x50 bolts.
- 3. Use allen key, located in top of post) to screw in bolts.
- 4. Align the foot, and screw in both bolts.
- 5. Make sure both bolts are properly fastened.



Important! Use the longest bolts (M10x50)when installing foot, shorter bolts may cause collapse.

Mounting boom to post

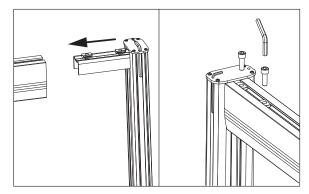


- 1. Slide washer and post in to boom.
- 2. Insert bolts trough holes and washer in boom.
- 3. Use allen key, (located in top of post) to screw in bolts.
- 4. Make sure both bolts are properly fastened




Do not stand under the boom when mounting, use a stand or similar to support the boom.

Mounting second post, trolley and end stop

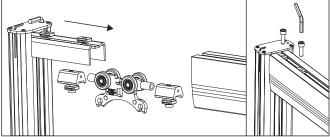

- 1. Slide end stop, trolley and end stop into boom in described order.
- 2. Slide washer and post in to boom.
- 3. Insert bolts trough holes and washer in boom.
- 4. Use allen key (located in top of post) to screw in bolts .
- 5. Make sure both bolts are properly fastened.

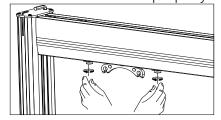


Fasten end stops temporary during assembly. Must be fastened properly with tools after adjustment of height.



Important! Fasten end stops to prevent trolley from moving when adjusting post heigts. Moving trolley can unintendedly hit the operator.

Assembly

Adjusting post heights

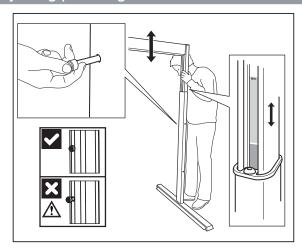

- 1. Lift the inner post and pull out quick release holt
- 2. Lift post to desired height and insert quick release bolt trough outer post.
- 3. Look at height labels and ensure that both post are adjusted at the same height.



Don't stand under boom when adjusting post height




Make sure quick release bolt is properly installed




Don't lift boom any higher when stop sign appears on height label

End stops

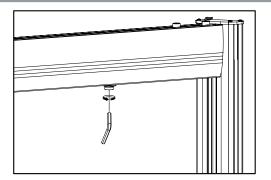

- 1. Lead the end stops to the desired position and tighten them.
- 2. Make sure the liftmotor does not hit the post.
- 3. Use allen key (located in top of post) to screw in bolts.

Mounting charger to post

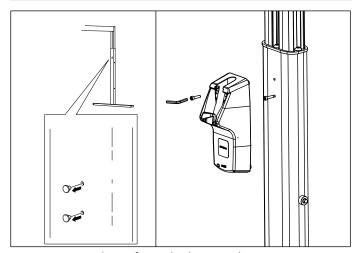

- 1. Remove plugs from holes on the post.
- 2. Use the small allen key (located in top of post) to fasten screws.
- 3. Fasten the lower screw first.
- 4. Place charger on screw and insert second bolt.

Check list before use




Don't start using MRS Duo before completing control according to checklist after assembly.

Use this checklist to verify that the MRS Duo is properly installed and can operate correctly and safely before use.

- ☐ Make sure both bolts in foot are properly fastened.
- ☐ Make sure all four bolts on boom are propearly fastened.
- ☐ Make sure quick release bolts are correctly installed
- ☐ Make sure end stops are properly fastened

How to use MRS Duo

Lifting user

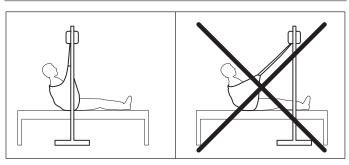

- 1. Make sure the floor is level.
- 2. Make sure the MRS Duo are stable, screw the adjustable screws in foot to correct position
- 3. Place the MRS Duo straight above the user.
- 4. Read the user manual for Molift Air or Nomad before lifting patient.




Never lift the user higher than necessary to carry out a lift.



Never leave a user unattended in a lifting situation.

Transport

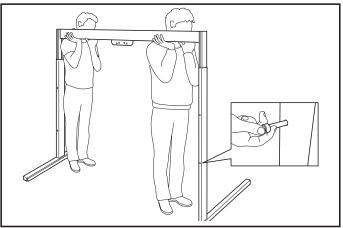

- 1. Secure trolley (se page 6, fastening end stop temporarily)
- 2. Remove quick release bolt and lower MRS
- 3. Place the quick release bolt in the transport hole (lowest hole)
- 4. Transportation height is now 1,7 m.
- 5. Move the MRS Duo as shown in illustration



Never transport MRS Duo without quick release bolt inserted. Post may fall apart.


Transport and storage

MRS Duo can be stored and transported under temperatures between -25 - 70 °C and Relative Humidity between 15 - 93 %.

Accessories

Battery charger


Molift Air or Molift Nomad ceiling hoist is charged through the hand control. The hand control charger can be mounted on the MRS Duo.

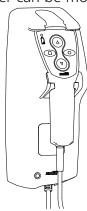

Charger for hand control Molift Nomad/AIR. Art. no.: 1240100

Molift Air

Molift Air 205 and Molift Air 300 is suitable for use with MRS Duo.

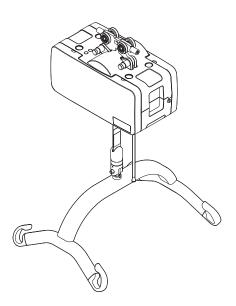

Molift Air is available with or without propulsion, with 205 or 300 kg SWL.

- Molift Air 205
- Molift Air 300
- Molift Air 205 Propulsion
- Molift Air 300 Propulsion


Trolley for Molift Air

MRS Trolley Basic

Art. no.: 2520000 MRS Trolley Propulsion Art. no.: 2500103

Molift Nomad

Molift Nomad portable hoist is suitable for use with MRS Duo.

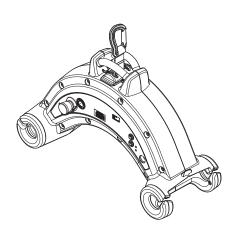

Molift Nomad is available with 160, 205, 230 or 255 kg SWL.


- Molift Nomad 160 kg SWL
- Molift Nomad 205 kg SWL
- Moilft Nomad 230 kg SWL
- Molift Nomad 255 kg SWL

Trolley for Molift Nomad

Trolley with brake

Art. no.: 1109109

Boom

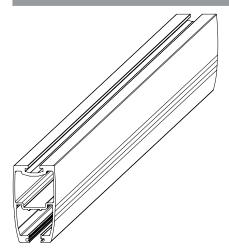

MRS Duo can be delivered with 4 different boom lengths from 2 - 3,5 meters.

Boom for MRS Duo:

Art. no.: 1600012-20 - 2 m Art. no.: 1600012-25 - 2,5 m Art. no.: 1600012-30 - 3 m Art. no.: 1600012-35 - 3,5 m

Maintenance

Cleaning and disinfection

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the lifter. For disinfection when needed; use isopropyl alcohol. Avoid abrasive cleaning products. Remove hair and pile from the castors and verify that the castors rotates freely.



Make sure not to damage or remove labels when cleaning.

Reconditioning

Follow cleaning and assembly procedure, complete periodic inspection and use checklist after assembly to recondition the lifter.

Recycling

MRS Duo can be disposed of as metal waste without battery charger. Battery charger must be removed and disposed of as Electrical equipment.

Spare Parts

A list of spare parts is available on request.

Periodic Inspection

Periodic Inspection Scope

Periodic inspection is a visual examination (particularly of the MRS Duos structure, bolts and lifters with attachments, safety devices and personsupport devices) according to Periodic Inspection Report for MRS Duo.



In the event of danger to safety, the MRS Duo shall immediately be taken out of service and marked clearly with "out of order" and shall not be used until the MRS Duo is repaired

Periodic Inspection shall be performed at least once a year or more frequently if required by local requirements. The inspection should be performed by service personnel authorized by Etac. Contact Etac for training and authorization or recommendation of an approved service partner.

When performing a periodic inspection, the inspector shall fill out the inspection report for MRS Duo. The reports should be retained by the person(s) responsible for servicing the lifter.

When periodic inspection is completed the inspector shall mark the MRS Duo with a sticker on the control label showing the date when periodic inspection is performed and a label (with 6-digit ID from Etac or distributor) showing who has performed the periodic control. The control label can be found on the post, and this will then indicate when next service should be performed.



Troubleshooting

Symptom	Possible Cause/Action
Wobbly between foot and post.	Check bolts in foot. Make sure bolts is properly fastened.
Wobbly between inner and outer post.	Check slider in outer post for wear. Change sliders.
Wobbly between boom and post.	Check bolts between bracket and boom. Make sure bolts are properly fastened.
	Controll bolts between bracket and post. Make sure bolts is properly fastened.
MRS Duo jams when lowering boom to transport position.	Quick release bolt is already inserted in bottom hole, inner post jams against bolt. Remove bolt.
	Lowering one post only, try to lover both post at the same time, or put quick release bolt in lowest user position, then start to lower the other side.
	Check slider in outer post for wear. Change sliders.
Molift Air or Molift Nomad hits post during use.	Adjust end stopper to correct position.



Etac A/S Parallelvej 3 DK-8751 Gedved

