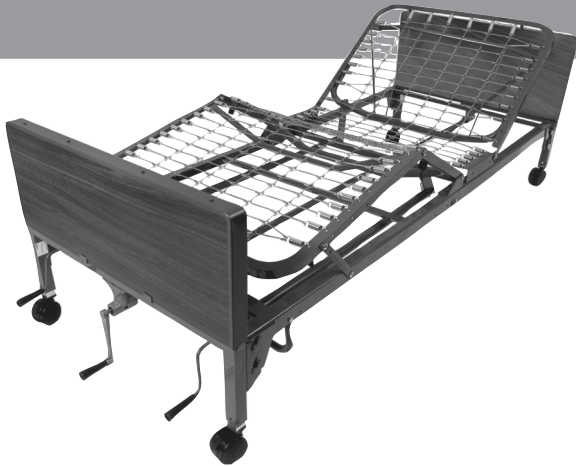
ELECTRIC AND MANUAL BEDS

OWNER'S ASSEMBLY AND OPERATING MANUAL

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Lit Manuel Manivelle Unique

15004

Semi-Electric Bed Single & Two Crank - Hi/Lo

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Manivelas – Alto/Bajo

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Manivelles – Hi/Lo

15005

Full Electric Bed

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SPECIFICATIONS

PACKAGING/HANDLING

UNPACKING

1. Check bed components for any obvious damage to the carton or its contents. If damage is evident, contact your local Drive dealer.
2. Remove any loose packing from the cartons.
3. Carefully remove all components from the cartons.

The three cartons and contents are as follows:

1. Bed End carton:

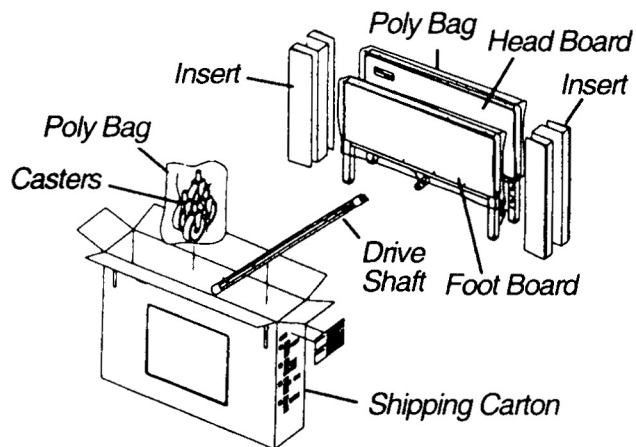
One (1) Head end.

One (1) Foot end (crank assembled on Bed No. 15003)

One (1) Drive Shaft Assembly-two-piece.

- * Four (4) Casters (two [2] locking and two [2] non locking casters).

NOTE: Set the Drive Shaft Assembly aside for Bed Nos. 15003, 15004, 15005

2. Foot Spring carton-Electric models (15012 or 15013):

One (1) Electric foot spring w/junction box and cord.

One (1) Hand pendant control.

One (1) Pendant holder.

One (1) Emergency crank handle.

CAUTION

DO NOT attempt to operate bed controls prior to completion of assembly as damage to bed components or personal injury may occur.

3. Head Spring carton:

One (1) Head spring

INSPECTION

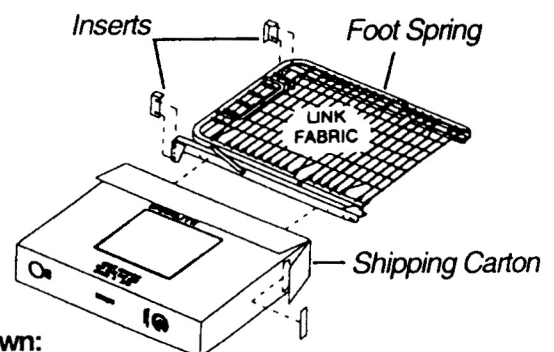
1. Examine each item carefully for scratches, nicks, dents or any other apparent damage.

NOTE: For semi-electric and full-electric beds:


2. Inspect the power supply cord for cuts or damage to the cord and/or plug.
3. Inspect the junction box for any damage to the connectors.
4. Ensure that the motor plugs are in good working order and fit properly into the junction box.

STORAGE

1. If bed is not to be assembled immediately, retain the cartons and packaging for storage until assembly is to be performed.
2. Store the repackaged Drive homecare bed in a dry area.
3. DO NOT place other objects on top of the repackaged cartons.

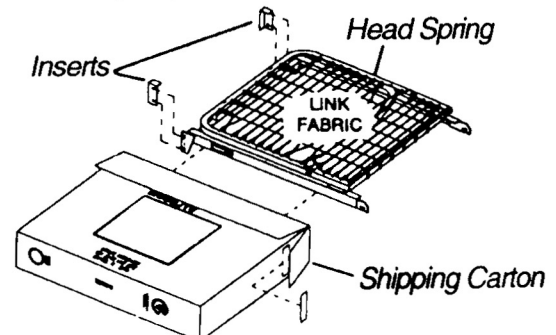Not Shown:
Junction Box w/Cord
Pendant and Emergency Crank Handle

ASSEMBLY

CONNECTING THE SPRING SECTIONS (SEMI/FULL ELECTRIC AND MANUAL BEDS) (FIGURE 1)

1. Remove the head spring (Model 15008) from packaged container and place on its side to your left with the center mounting latches to your right.
 2. Remove the foot spring from packaged container and place on its side to your right with the head spring pull tube at the top of the assembly and the center mounting rivets facing to the left.
 3. For semi-electric (15012) and full-electric (15013) foot spring, the motors should be to your right and for manual foot spring (15009) the crank handles will be to your right.
 4. The head and foot spring should now be on their side at approximately a 90° angle to one another.
 5. Hook the bottom head spring center mounting latches to the bottom foot spring center mounting rivets.
 6. Hook the top head spring center mounting latches to the top foot spring center mounting rivets.
- NOTE: It may be necessary to lift slightly on the head spring or foot spring to secure the mounting latches and rivets.*
7. After head and foot springs are connected, push the springs into a horizontal (straight) position while still on their sides.

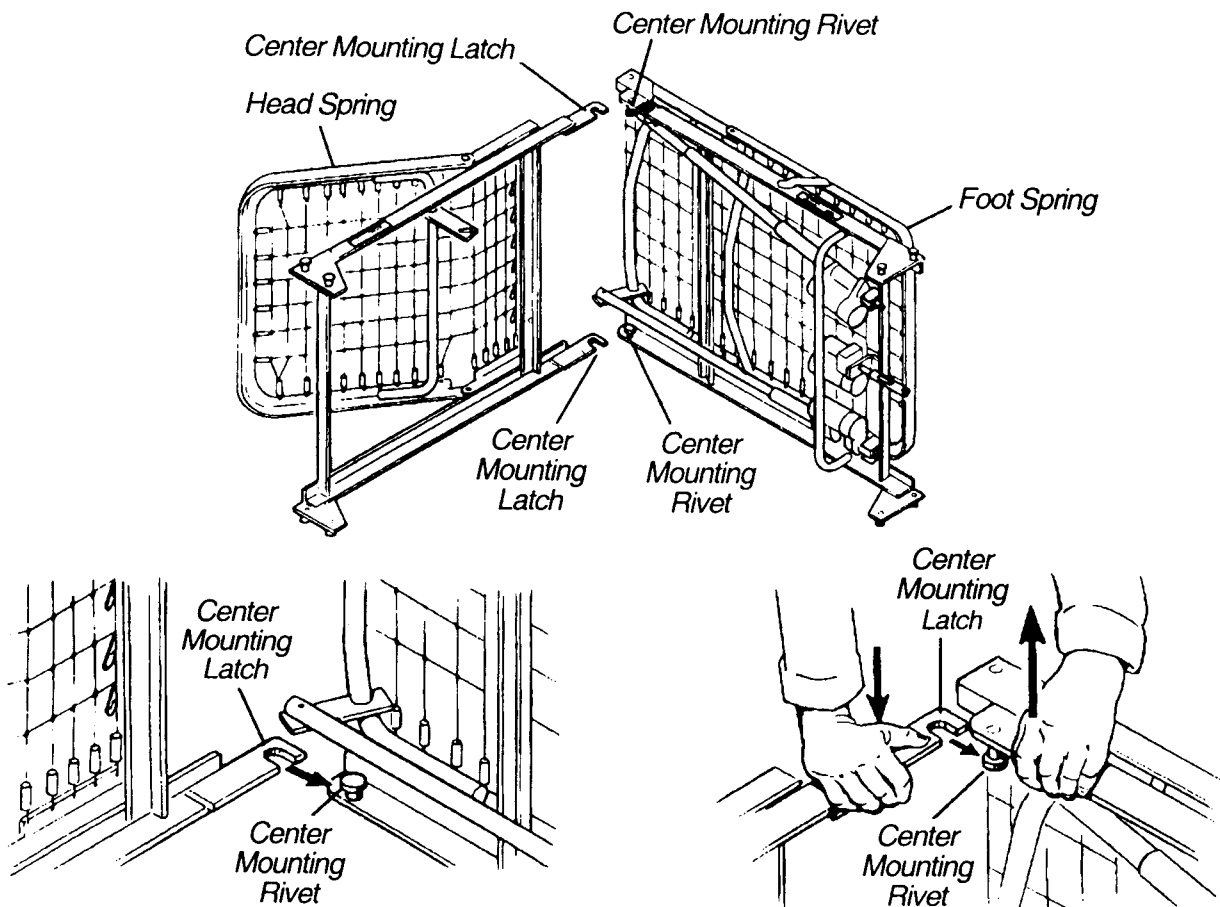
FIGURE 1 - CONNECTING THE SPRING SECTIONS

ASSEMBLY

CONNECTING THE SPRING FABRIC (FIGURE 2)

1. Lift the head spring upward to give slack to the fabric, and connect the spring fabric together with the links provided.

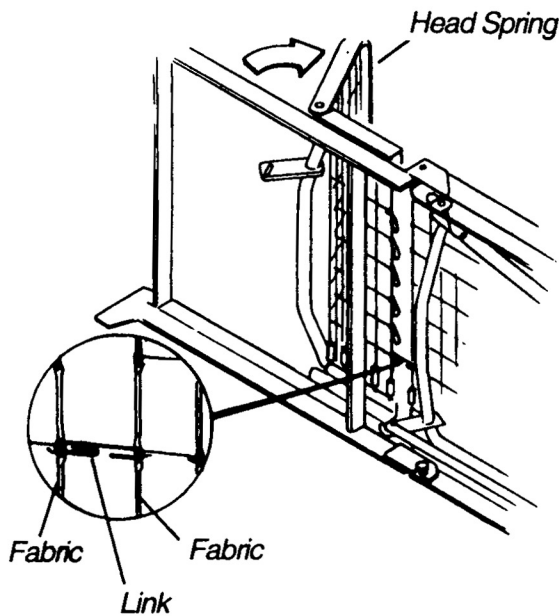
FIGURE 2 - CONNECTING THE SPRING FABRIC

ADJUSTING AND RECONNECTING THE HEAD SPRING PULL TUBE (FIGURE 3)

1. To provide up/down movement of the head section, disconnect the head pull-tube assembly from the side frame of the foot spring by removing the hitch pin, grommet and washer from the clevis pin.
2. Extend the inner pull tube shaft manually until the spring button "clicks" into the adjustment hole of the outer pull tube.
3. The spring button should protrude through the adjustment hole and the clevis pin of the now extended pull tube arm will line up with the lift arm slot.
4. Connect the pull tube end assembly to the lift arm of the head spring by inserting the clevis pin into the lift arm slot; reconnect the grommet and washer. Secure with hitch pin.

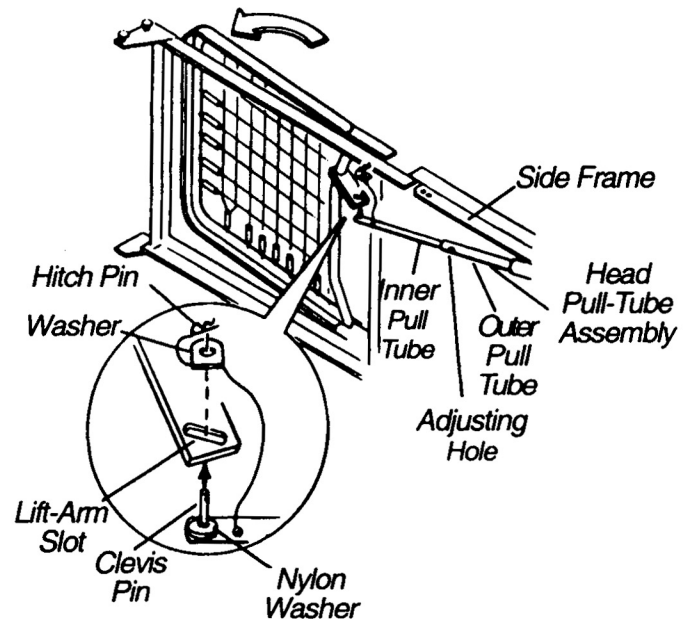
FIGURE 3 - ADJUSTING AND RECONNECTING THE
HEAD SPRING PULL TUBE

ASSEMBLY

ASSEMBLING THE BED ENDS (FIGURE 4)

NOTE: When connecting the bed ends (headboard and footboard), remember that the headboard is the taller of the two.

1. Stand the headboard as close to the head spring as possible.
2. With one hand, grasp the head spring frame and raise it until the rivets on the corner plates of the spring are at the correct height to fit into the corner locks on the headboard.
3. Tilt the headboard backward slightly and the rivets should slide into the corner locks more easily.
4. When the headboard is returned to its full upright position, the headboard will lock into place.
5. Repeat the same procedure for the foot end section.

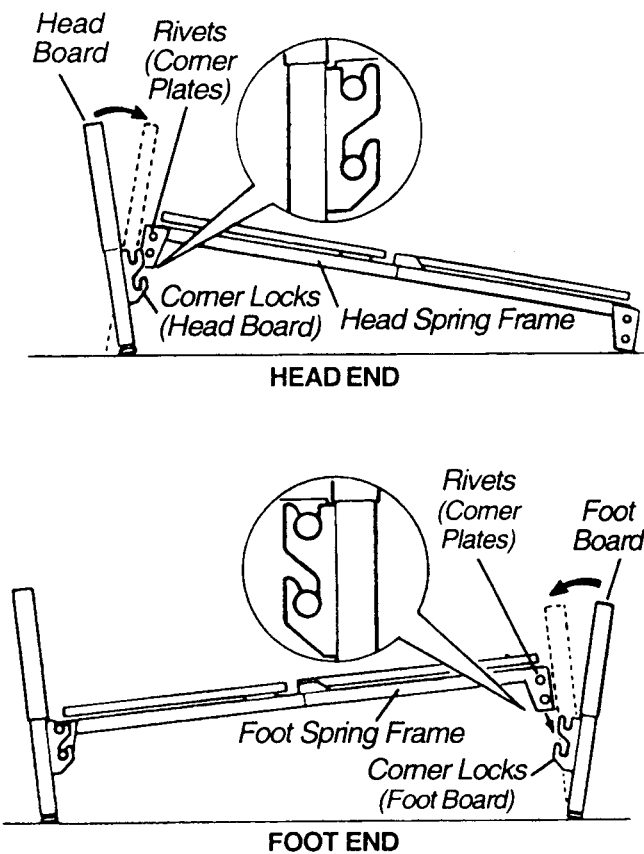
FIGURE 4 - ASSEMBLING THE BED ENDS

ASSEMBLY

INSTALLING THE CASTERS (FIGURE 5)

1. Install casters by carefully lifting the bed ends approximately 5-inches (12.7 cm) from the floor and insert the shaft of caster assembly into the caster socket.

NOTE: Position the two (2) locking casters diagonally opposite from one another.

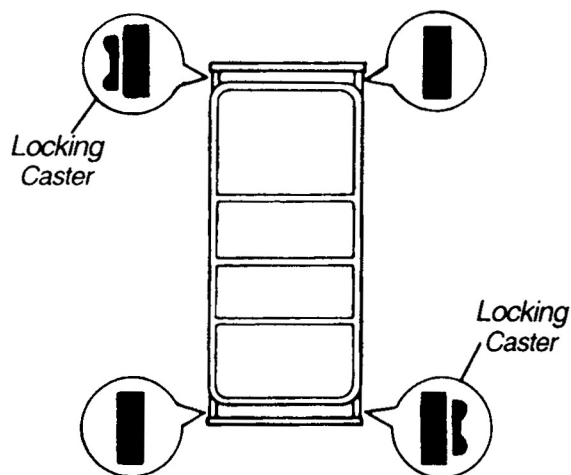
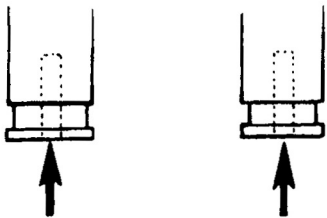
FIGURE 5 - INSTALLING THE CASTERS

DRIVE SHAFT ASSEMBLY AND INSTALLATION (FIGURE 6)

The drive shaft is composed of two (2) sections. The inner shaft which has a positioning spring button, and the outer shaft, which has two (2) positioning holes and one (1) storage position hole. Together they make the telescoping drive shaft used on all manual and electric Hi/Lo model beds.

1. Remove the plastic caps from each end of the drive rod.
2. Depress spring button on the inner shaft and insert the inner shaft into the outer shaft.
3. Slide the inner shaft to the desired position for the type of bed being used and engage the spring button into the proper positioning hole.
4. For semi-electric and manual beds, engage the spring button into the furthestmost positioning hole in outer shaft.
5. For full-electric models, engage the spring button in to the positioning hole in the outer shaft located next to the semi-electric/manual positioning hole.
6. Connect the drive shaft to the bed by attaching either end of the drive shaft to the head end gear box.
7. Connect the other end to the Hi/Lo motor output shaft and then connect the foot end gearbox output shaft to the Hi/Lo motor output shaft spring-loaded coupler.
8. Release the drive shaft spring-loaded coupler by pushing in against the spring-loaded coupler and turn it clockwise.
9. The Hi/Lo motor output shaft spring-loaded coupler must now be released to properly connect with the foot end gear box output shaft.
10. Do this by pushing in against the Hi/Lo motor output shaft spring-loaded coupler and turn it clockwise.
11. The coupling will release and engage the foot end output shaft.
12. Ensure that all components are securely installed.

ASSEMBLY

NOTE: Since there is no Hi/Lo motor for semi-electric Single Crank, Two Crank and manual beds with Hi/Lo functions, the drive shaft will mount directly into the foot end gearbox output shaft.

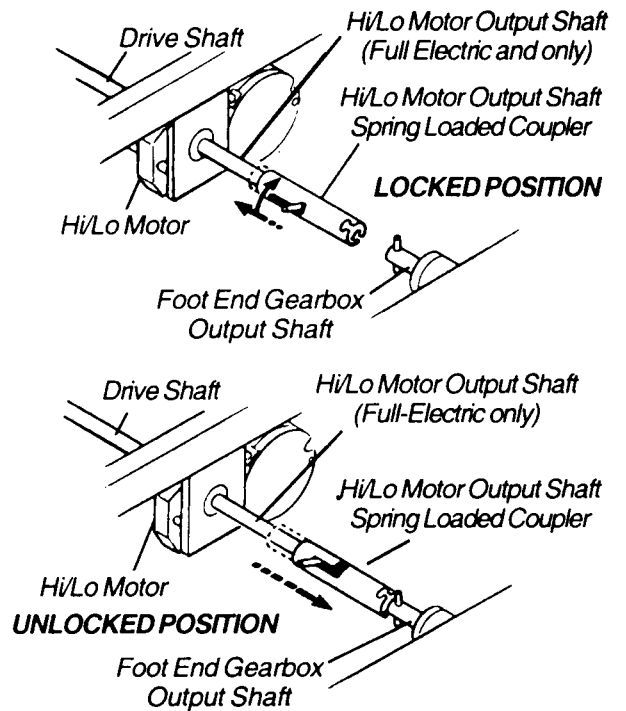
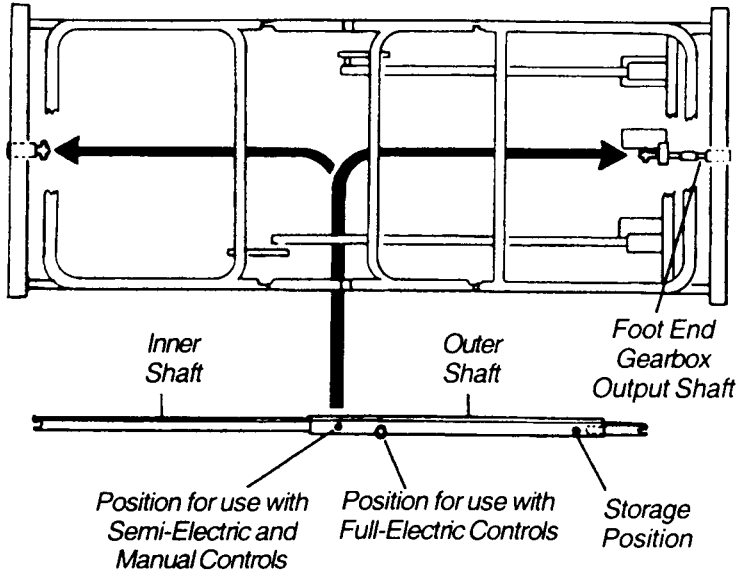
FIGURE 6 - DRIVE SHAFT ASSEMBLY AND INSTALLATION

ACTIVATING THE BED FUNCTIONS (FULL/SEMI ELECTRIC BEDS ONLY) (FIGURE 7)

1. Recheck all electrical connections on junction box at foot end ensuring that all motors are plugged into the correct locations.
2. Plug the junction box cord into a properly grounded 110-volt, 60 cycle outlet.
3. Install pendant holder in convenient place for user.

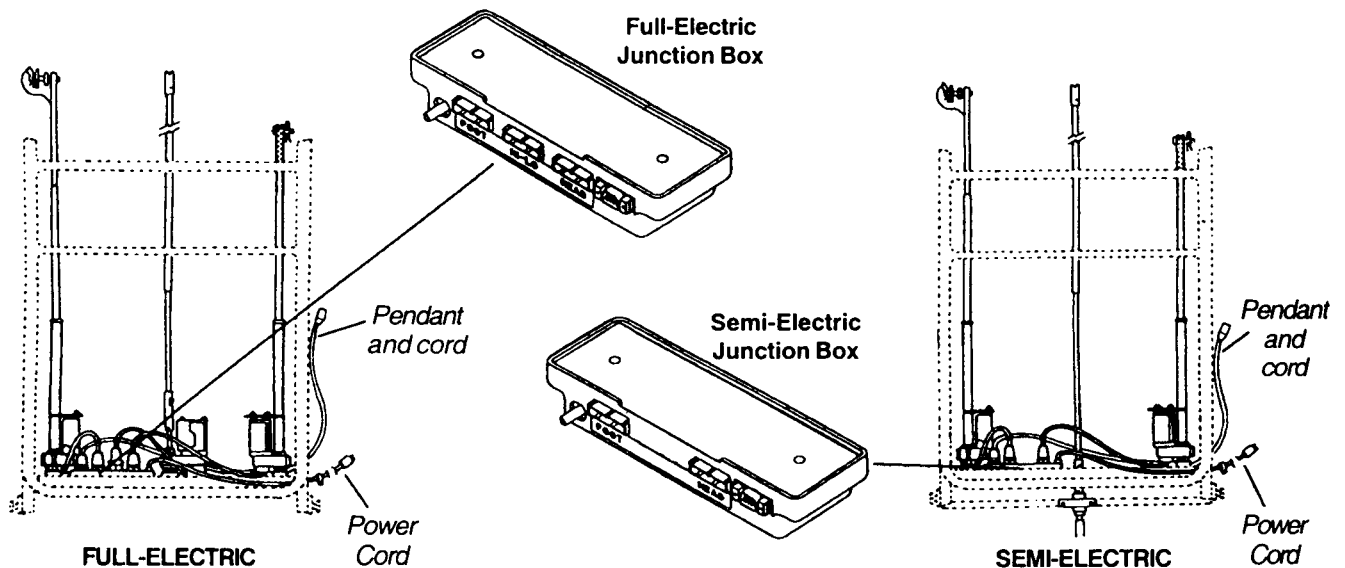
FIGURE 7 - ACTIVATING THE BED FUNCTIONS (FULL/SEMI ELECTRIC BEDS ONLY)

OPERATION AND ADJUSTMENT

This Procedure Includes the following:

Full Electric Operation

Semi-Electric Operation

Manual Bed Operation

FULL-ELECTRIC OPERATION

1. The accompanying pictures detail the full-electric beds and will acquaint the user with the bed functions and controls.
2. Operate the bed through all phases of its operation.
3. If any problems arise during test, recheck all electrical connections and mechanical hook ups.
4. Full-Electric beds use a six (6) function pendant for all bed operation.
5. All Full-Electric beds come with an emergency crank to allow continued operation in the event of a power outage.


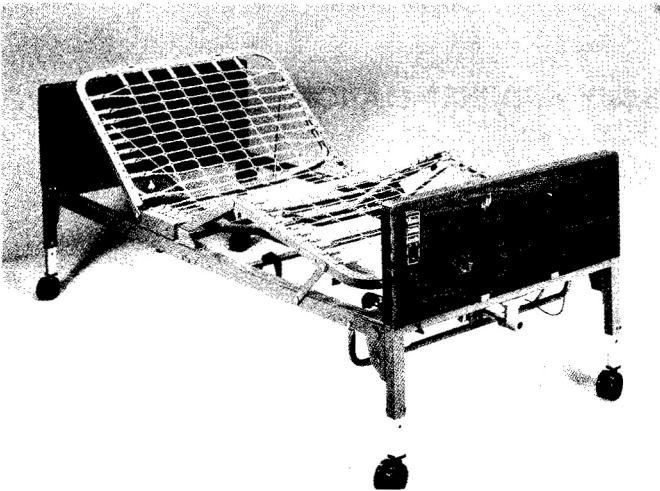
FIGURE 1 - FULL-ELECTRIC BED

Raising and Lowering the Entire Bed (Full-Electric Beds [FIGURE 2])

TO RAISE ENTIRE BED - FULL ELECTRIC BEDS

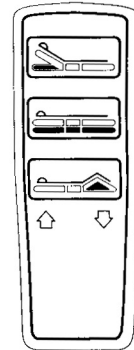Depress button "BED" "UP" as shown on pendant control. This will raise the entire height of the bed.

TO LOWER ENTIRE BED - FULL ELECTRIC BEDS

Depress button "BED" "DOWN" as shown on pendant control. This will lower the entire height of the bed.

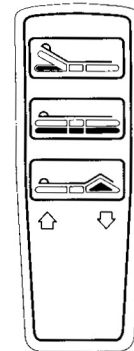
FIGURE 2 - RAISING AND LOWERING THE ENTIRE BED (FULL-ELECTRIC BEDS)

OPERATION AND ADJUSTMENT

Raising and Lowering the Head and Foot Sections (Full-Electric Beds [FIGURE 3])

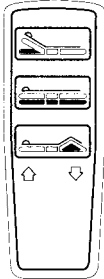
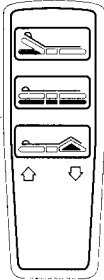
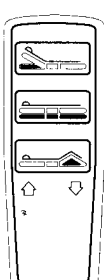
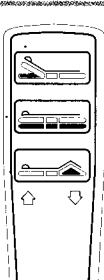
<p>TO RAISE THE HEAD SPRING SECTION FULL ELECTRIC BEDS</p>	
<p>Depress button "HEAD DOWN" as shown on pendant control. This will lower the head portion of the bed ONLY.</p>	
<p>TO LOWER THE HEAD SPRING SECTION FULL ELECTRIC BEDS</p>	
<p>Depress button "HEAD DOWN" as shown on pendant control. This will lower the head portion of the bed ONLY.</p>	
<p>TO RAISE THE FOOT SPRING SECTION FULL ELECTRIC BEDS</p>	
<p>Depress button "FOOT UP" as shown on pendant control. This will raise the foot portion of the bed ONLY.</p>	
<p>TO LOWER THE FOOT SPRING SECTION FULL ELECTRIC BEDS</p>	
<p>Depress button "FOOT DOWN" as shown on pendant control. This will lower the foot portion of the bed ONLY.</p>	

FIGURE 3 - RAISING AND LOWERING THE HEAD AND FOOT SECTIONS (FULL-ELECTRIC BEDS)

SEMI-ELECTRIC OPERATION

1. The accompanying pictures detail the semi-electric beds with the hi/lo and foot crank locations where applicable.
2. The semi-electric Single and Two Crank beds with Hi/Lo operation use a four (4) function pendant for head and footspring functions and a hand crank for hi/lo bed operation.
3. The semi-electric fixed height beds will feature head and foot section operation but the overall bed height is fixed and cannot be altered.
4. Determine your particular semi-electric bed to assist in the operation of the features of the bed.
5. Operate the bed through all phases of its operation.
6. If any problems arise during test, recheck all electrical connections and mechanical hook ups.
7. All semi-electric beds come with an emergency crank to allow continued operation in the event of a power outage.


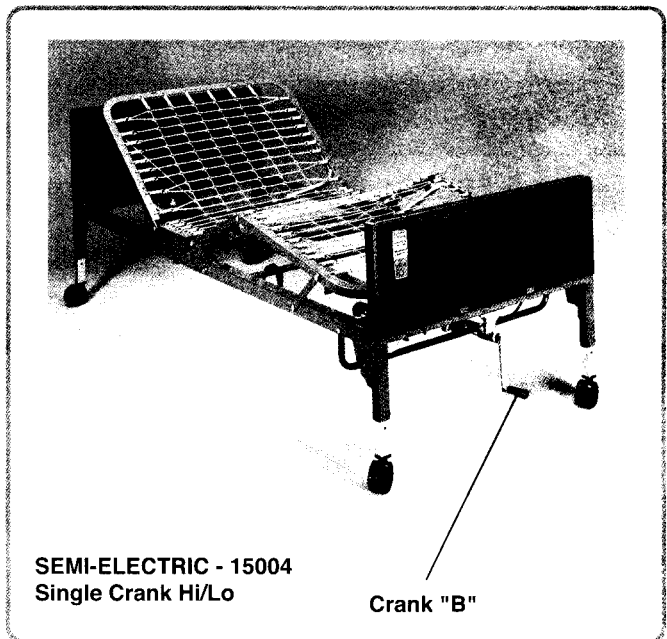
FIGURE 4 - SEMI-ELECTRIC OPERATION

OPERATION AND ADJUSTMENT

Raising and Lowering the Entire Bed (Semi-Electric Beds [FIGURE 5])

TO RAISE ENTIRE BED-SEMI-ELECTRIC BEDS

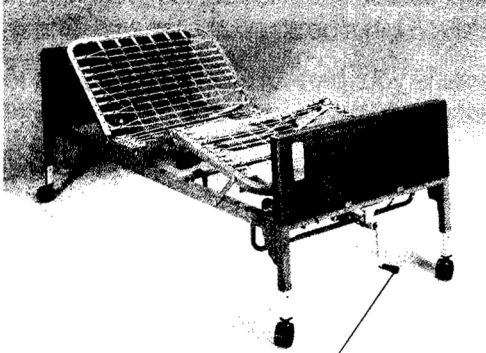
SEMI-ELECTRIC SINGLE CRANK-HI/LO:
Turn crank "B" clock wise. This will raise the entire height of the bed.

Crank "B"

TO LOWER ENTIRE BED - SEMI-ELECTRIC BEDS

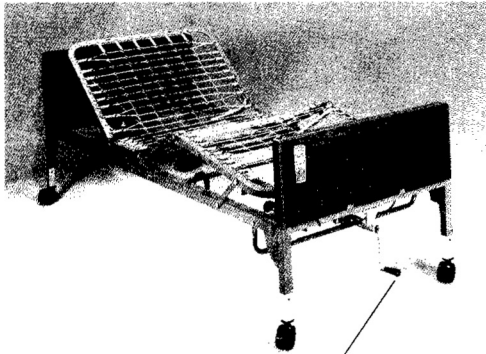SEMI-ELECTRIC SINGLE CRANK-HI/LO:
Turn crank "B" counter clockwise. This will lower the entire height of the bed.

Crank "B"


FIGURE 5 - RAISING AND LOWERING THE ENTIRE BED (SEMI-ELECTRIC BEDS)

Raising and Lowering the Head and Foot Sections (Semi-Electric Beds [FIGURE 6])

TO RAISE HEAD SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

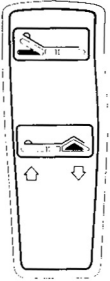
Depress button "HEAD UP" as shown on pendant control. This will raise the head portion of the bed ONLY.

TO LOWER HEAD SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

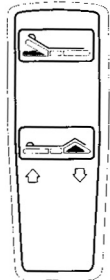
Depress button "HEAD DOWN" as shown on pendant control. This will lower the head portion of the bed ONLY.

TO RAISE FOOT SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

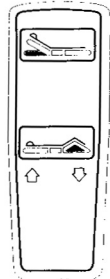Depress button "FOOT UP" as shown on pendant control. This will raise the foot portion of the bed ONLY.

TO LOWER FOOT SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

Depress button "FOOT DOWN" as shown on pendant control. This will lower the foot portion of the bed ONLY.

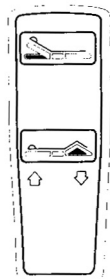
FIGURE 6 - RAISING AND LOWERING THE HEAD AND FOOT SECTIONS(SEMI-ELECTRIC SINGLE AND TWO CRANK HI/LO BEDS)

NOTE: Remember that in the event a full or semi-electric bed should malfunction, it is possible to manually crank the bed using the emergency crank supplied.

OPERATION AND ADJUSTMENT

MANUAL BED OPERATION (FIGURE 7)

1. The accompanying pictures detail the manual bed models and the hi/lo crank locations where applicable.
2. Determine your particular manual bed to assist in the operation of the features of the bed.
3. Operate the bed through all phases of its operation.
4. If any problems arise during test, recheck all mechanical hook ups.


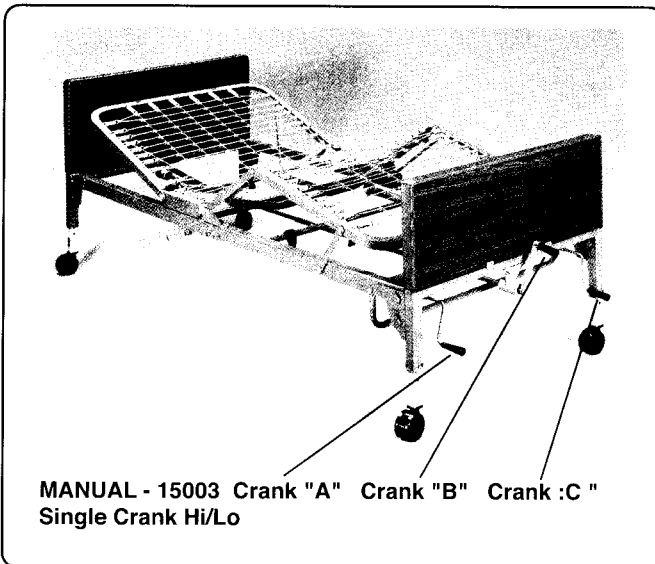
FIGURE 7 - MANUAL BED OPERATION

Raising and Lowering the Entire Bed (Manual Beds [FIGURE 8])

TO RAISE ENTIRE BED - MANUAL BEDS

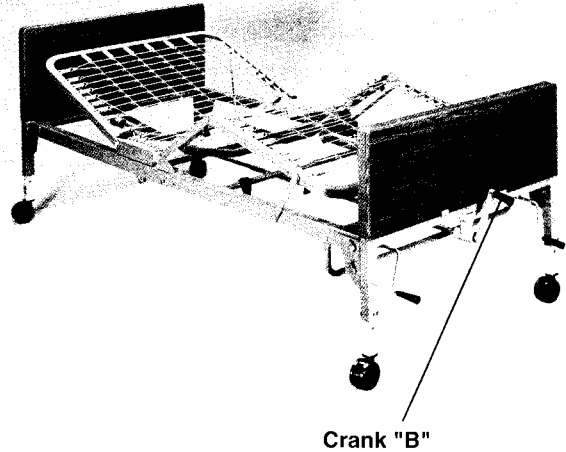MANUAL SINGLE CRANK-HI/LO:
Turn crank "B" clockwise. This will raise the entire height of the bed.

TO LOWER ENTIRE BED - MANUAL BEDS

MANUAL SINGLE CRANK-HI/LO:
Turn crank "B" counterclockwise. This will lower the entire height of the bed.

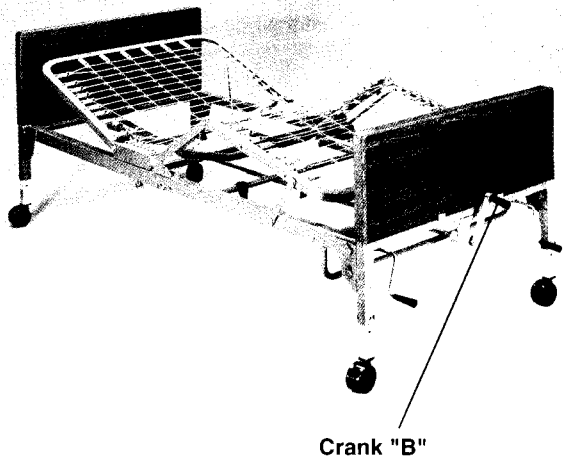
FIGURE 8 - RAISING AND LOWERING THE ENTIRE BED (MANUAL BEDS)

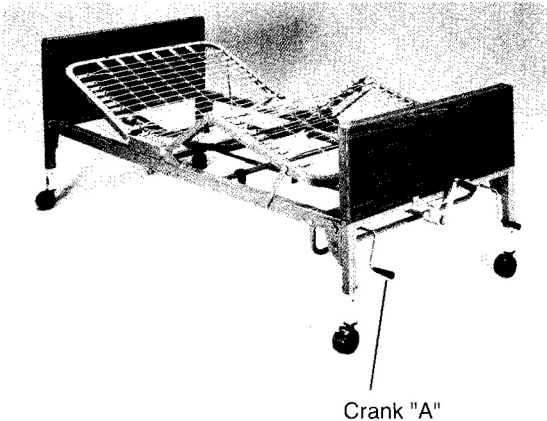
OPERATION AND ADJUSTMENT

Raising and Lowering the Head Section (FIGURE 9 A)

TO RAISE HEAD SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

Turn crank "A" clockwise. This will raise the head portion of the bed ONLY.

TO LOWER HEAD SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

Turn crank "A" counterclockwise. This will lower the head portion of the bed ONLY.


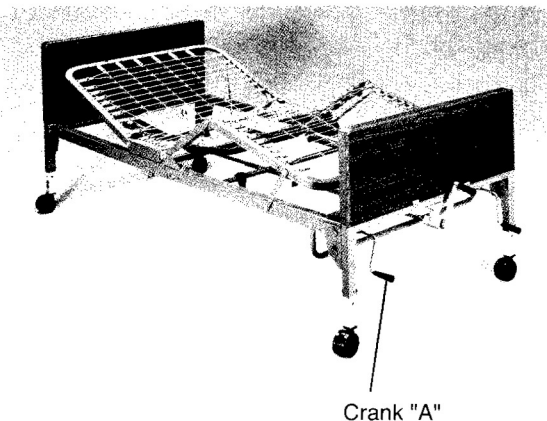
FIGURE 9A - RAISING AND LOWERING THE HEAD SECTION

Raising and Lowering the Foot Section (FIGURE 9 B)

TO RAISE FOOT SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

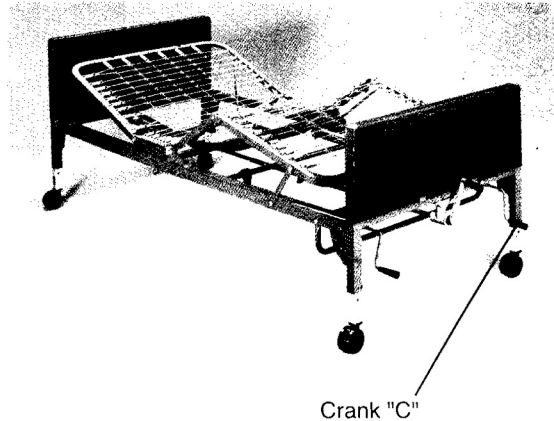Turn crank "C" clockwise. This will raise the foot portion of the bed ONLY.

TO LOWER FOOT SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

Turn crank "C" counterclockwise. This will lower the foot portion of the bed ONLY.

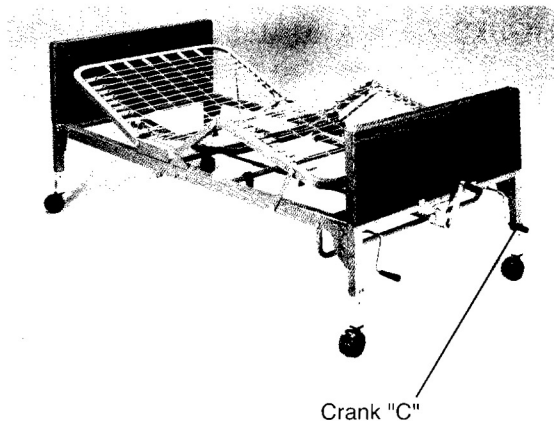
FIGURE 9B - RAISING AND LOWERING THE FOOT SECTION

MAINTENANCE AND SAFETY CHECKS

Drive recommends the following maintenance and cleaning procedures be conducted between users.

ELECTRONICS

- Check all controls to make sure all functions work properly.
 - Foot control
 - Head control
 - Hi/LO
- Check all cables for damaged or frayed wires.
 - Power cord
 - Pendant cord
- Check to make sure all plugs are fully inserted or attached.

Bed frame and sleeping surface

- Visually check all welds
 - Head section
 - Foot section
 - Main Frame
- Check joints between sleeping surface sections for loose fasteners.

Cleaning

- The metal parts of the bed are covered with a powder coating. Clean all coated parts with mild detergent and warm water. Periodically raise head and feet sections of the bed and remove dust from frame. Also, periodically remove mattress and clean mattress deck.

Lubrication and Mechanical

- Lightly grease all actuator screw threads with white lithium grease.
- Lubricate all caster roller and swivel bearings with light machine oil.
- Check all bolts and tighten as needed.

WARRANTY

Lifetime warranty on welds.

Five year warranty on frame.

One year warranty on all other parts and components.

During the warranty period, defective items will be repaired or replaced at Drive's option at no charge.

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