



The JAZZ

Let **Jazz** be your new dance partner. Dolomite's new rollator is stylish and will make you the talk of the dance floor. A beautiful charcoal color frame is lightweight and the unique side to side, cross-fold makes it easy to fold and store. **Jazz** is designed so that you walk in front of the back wheels. This gives you more support and stability by putting wheels both in front and behind you.

Components

Jazz is constructed with a welded aluminum frame for weight savings. A welded frame is less maintenance than a frame put together with screws and nuts. The large 8" wheels are suitable for indoor or outdoor travel. A comfortable cloth seat has a loop that will release the safety catch to fold. Just lift up the loop and the **Jazz**, including the flexible basket, quickly folds to 10" wide. Anatomic grips offer comfort for arthritic hands and promote proper grip positioning. The infinite handle height adjustment has markings to assist in duplicating a handle setting.

Brake cables are integrated in the frame to stay out of the way when walking or folding. A curb climber pedal on the rear fork makes it easier to lift the front wheels over obstacles. The Dolomite brake handles operate both the wheel brakes and set a parking brake when seated. A mesh fabric basket, standard, has a wire frame and can stay in place when **Jazz** is folded.

Jazz's Folding Feature

The new cross fold action of the **Jazz** is easy to do, simply pull up on the red ribbon on the seat. The **Jazz** will unlatch and smoothly fold. When folded the **Jazz** will stand upright and the small profile can be easily stored in your home.



Jazz folding

CLARKE HEALTH CARE PRODUCTS, INC.

1003 International Drive • Oakdale, PA 15071

Phone (888) 347-4537 • Fax (724) 695-2922 • www.clarkehealthcare.com





The JAZZ

Accessories

Features

- Two frame heights
- Comfortable fabric seat
- Easy to fold, stores standing upright
- Safety latch secures open frame
- Flexible back support strap
- Spacious fabric basket



BASKET



CURB CLIMBER



RELEASE LATCH

Accessories

To complete your **Jazz** partner, an acrylic tray is available. It rests on the seat to carry beverages or personal articles.



TRAY

Warranty

Jazz is covered by a two year limited warranty against manufacturer's defects. This warranty excludes wear and tear parts such as wheels, hand grips and brake cables.

Specifications

SPECIFICATIONS	STANDARD	LOW
Basket Capacity	10 lbs	10 lbs
Wheel	8"D x 1 1/4" W	8"D x 1 1/4" W
Capacity	287 lbs	287 lbs
Weight of Walker	17 lbs	15 3/4 lbs
Overall Width	24"	24"
Overall Length	26"	26"
Width Between Wheels	17"	17"
Overall Height	29 1/2" - 37 3/4"	26" - 31 1/2"
Seat Height	23 1/2"	20"
Seat	14 1/2" L x 8" W	14 1/2" L x 8" W
Width Between Handles	15"	15"
Folded Width	10"	10"

CLARKE HEALTH CARE PRODUCTS, INC.

1003 International Drive • Oakdale, PA 15071

Phone (888) 347-4537 • Fax (724) 695-2922 • www.clarkehealthcare.com

