

## 6810A: DROP ARM VERSAMODE (WITH COMMODE PAIL) ASSEMBLY INSTRUCTIONS



Warning: This product should not be used without proper instruction from a healthcare professional.



Warning: Lumex assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.




Warning: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.

## Assembly instructions (see picture at right)

- 1. Install back & seat assembly on base frame assembly.
- 2. Tighten and fix left and right knobs.
- 3. Install left and right armrests. Lock into place.
- 4. Install left and right footrests on base frame assembly brackets. Lock into place.
- 5. Slide the commode set under the seat set from back side.



Warning: Ensure that all components are securely installed as described above before use.

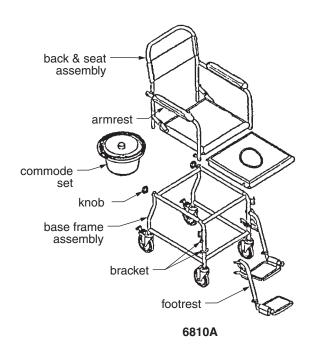

## WARRANTY

Lifetime limited warranty. For additional warranty information, please contact your Lumex dealer.

Graham-Field Health Products Atlanta, Georgia 30360

